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## SpeVaX<sup>®</sup> – Instruction for Use

### *Trach-HME with Built-in Speech Valve*

**Manufacturer:** **BenCorpp<sup>®</sup> Srl**  
Via Imperiale, 279  
IT - 41037 – Mirandola (MO)  
[info@bencorpp.com](mailto:info@bencorpp.com)

### **Indications**

Heat & moisture exchanger for patients breathing spontaneously via an ET tube or a tracheostomy tube in the hospital or at home. SpeVaX<sup>®</sup> is recommended for patients weighing no less than 10 kg.

### **Target patient group**

Target patient group is represented by patients with permanent tracheostomies, Intensive Care patients with artificial airways (spontaneously breathing) and children (not less than 10 kg) with artificial airways (spontaneously breathing) in a hospital or home environment.

### **Intended users**

Lay users (home use) and physicians (hospital environment).

### **Expected clinical benefits**

- Reduce coughing and mucus formation.
- Reduce the transmission of microbes and other particulate matter.

### **Description**

SpeVaX<sup>®</sup> is a heat and moisture exchanger with speech valve and connector for supplemental oxygen supply that ensure effective humidification and warming of inspired air for tracheostomy patients, can be fitted to the standardized connectors of tracheostomy tubes or endotracheal tubes, SpeVaX<sup>®</sup> does not come into direct contact with the patient, it is supplied in carton fifty or hundred pieces.

### **Connections**

SpeVaX<sup>®</sup> is equipped with a 15mm female connector for ET tube or tracheostomy tube and a 4mm connector for supplemental oxygen supply. SpeVaX<sup>®</sup> has a valve for forced inspiration/expiration or suctioning.

### Technical specification

<b>Recommended tidal volume range</b>	50 - 1000 ml
<b>Compressible volume</b>	9,5 ml (dead space)
<b>Moisture output</b>	V <sub>T</sub> = 500 ml: 13,1 mg/l
<b>Moisture loss (24 h)</b>	min 7,5 ± 0,3 mg/l; max 19,4 ± 0,2 mg/l
<b>Pressure drop (24 h)</b>	30 l/min = 34 ± 4 Pa 60 l/min = 113 ± 24 Pa 90 l/min = 218 ± 35 Pa
<b>Weight</b>	2,9 g
<b>Length</b>	22 mm
<b>Height</b>	28 mm
<b>Width</b>	38 mm
<b>Connectors</b>	15 mm female connector for ET tube or tracheostomy tube. 4 mm connector for supplementary oxygen supply.
<b>Non-conductive</b>	

### Instructions for use

The device should be placed at the open end of the ET tube or tracheostomy tube. Check that the connection is properly fixed. Normal efficiency is reached after a few breaths. Should be replaced every 24 hours or more frequently, to avoid an increase in resistance due to accumulation of secretions.

Before use, check product for deformity and valve function by closing the valve and releasing it. Do not use if the device shows any signs of defect. Do not use if the package is opened or damaged.

### Storage:

Store in clean, dry, temperate conditions & keep away from sunlight. Temporary deviations are allowed within the temperature range (2- 50°C).

### Fixation the device

Hold the device with your index finger and thumb and gently press inward until the device connects to the tube.

### Speech valve (Speaking)

SpeVaX® contains a valve with a spring that can easily be occluded with a finger facilitating speech. After releasing the finger, the valve will open automatically.

### Removal the device

Hold the device and begin rotating it carefully, pulling outward until the device separates.

### Contraindications

Do not use beyond recommended tidal volume range, as the added dead space may cause CO<sub>2</sub> retention at too low tidal volumes. A too high tidal volume may lead to unsatisfactory humidification. Do not use on dehydrated patients or patients with very heavy secretions from the lungs and airways, or in combination with external humidifiers or nebulizer.

### Warning and precautions

Non-sterile, single use only: do not sterilize or reuse, as the intended function will be affected, and this might increase the risk of possible transmission of infection. Do not use the product after the expiration date. The expiry date refers to the intact and correctly stored product.

Destroy after use. Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

In case of undesirable effects, discontinue use, consult your doctor, and inform the manufacturer.

### Reporting

In the event of serious accidents occurring related to the use of the device SpeVaX® Trach-HME with Built-in Speech Valve, send a report to the manufacturer (info@bencorpp.com), and the national authority of the country in which the user and/or patient resides, if the product has been used in Kingdom of Saudi Arabia, please contact Saudi Food and Drug Authority, you can also contact the official agent via email: [info@medcaloasisint.com](mailto:info@medcaloasisint.com) / Tel: 00966 11 4037000

For home use: for further information contact the above agent within the KSA who provides technical assistance.

These Instructions for Use are also available on the manufacturer's website: <https://www.bencorpp.com/>

**GRAPHIC SYMBOLS CONTAINED IN DEVICE LABELING**

Consult Instructions for Use

Cautions on instructions for use

Manufacturer

Medical Device

Batch Code

Do Not Re-use

Catalog Number

Keep Away from Sunlight

Use By

Keep dry

Unique Device Identifier

Do not use if package is damaged

Store in temperate conditions.  
Temporary deviations within the  
temperature range (max-min) are  
allowed

Storage temperature limit